



DIGITAL SPECIALTYS

Office Address:-

Shop No-3, Block-D/1, Rajendra Vihar Apartment
Forest Park, Bhubaneswar-751009, Odisha, India

Factory Address:-

Plot No-359/2436, Ground Floor
Near Sukanti Devi Enclave, AIIMS Nagar Lane-1
Patrapada, Aaiginia, Bhubaneswar-751019, Odisha, India

PRODUCT SPECIFICATION

PRODUCT NAME : LATEX EXAMINATION GLOVES NON STERILE POWDERED SMOOTH (5gm.)

Special Features

- Quantity Assurance : 100 pcs packed in each box
- AQL 2.5 Assurance : Accepted Quality Limit of 2.5 followed
- Weight Assurance : 4.8 ± 0.2 gms per pc (medium size)
- Shelf Life Assurance: Antioxidant added to increase the shelf life.

Product Information

- Material : Natural Rubber Latex
- Colour : Pale Yellow, the color may vary due to storage time and conditions.
- Design : Straight Fingers, Thumb and Fingers in one plane, Fits either hand (Ambidextrous) Rolled Rim.
- Powder Content : <10.0 mg/dm² max (Inside absorbable cornstarch USPXX1)
- Sterilization : Non Sterile.
- Labeling : Shall comply with the corresponding labeling specification and customer requirement.
- Shelf Life : 3 years from the date of manufacture.
- Storage condition : Shall be stored in cool dry place and away from direct light.

Packaging Details

Inner Box	100 pcs
Master Carton	2000 pcs (100 x 20 inner boxes)

Certifications : CE
: ISO 9001 : 2015
: ISO 13485: 2016



DIGITAL SPECIALTYS

Office Address:-

Shop No-3, Block-D/1, Rajendra Vihar Apartment
Forest Park, Bhubaneswar-751009, Odisha, India

Factory Address:-

Plot No-359/2436, Ground Floor
Near Sukanti Devi Enclave, AIIMS Nagar Lane-1
Patrapada, Aaiginia, Bhubaneswar-751019, Odisha, India

Physical Dimensions

Length(mm) Min	Size	XS	S	M	L	XL
	Length	Min 240mm				
Width (mm)	Size	XS	S	M	L	XL
	Width (± 10)	70 ± 10	80 ± 10	95 ± 10	110 ± 10	120 ± 10
Thickness(mm) Min						
	Palm	Min 0.08mm (Single Wall)				
	Finger	Min 0.08mm (Single Wall)				

Physical Properties

Characteristics	Before Ageing	After Ageing (70 \pm 2°C for 166 \pm 2 hrs)
Tensile Strength (Mpa) min.	18 mpa Min	14 mpa Min
Ultimate Elongation (%) min.	650% Min	500% Min

Directions For Use:

1.1 Preparation for use : Ensure selection of gloves with correct size.

- Below is the correct glove size width :

Size	XS	S	M	L	XL
Width Range(mm)	70 ± 10	80 ± 10	95 ± 10	110 ± 10	120 ± 10

1.2 How To Use

- Dry hands before donning the gloves.
- Take out the product after checking the package.
- Gloves need to be donned carefully; do not apply excessive force to pull the gloves when donning.
- Keep the fingernails short and change gloves frequently to avoid tearing issue.
- The gloves should be discarded with an appropriate waste disposal container after use according to local disposal regulations.



DIGITAL SPECIALTYS

Office Address:-

Shop No-3, Block-D/1, Rajendra Vihar Apartment
Forest Park, Bhubaneswar-751009, Odisha, India

Factory Address:-

Plot No-359/2436, Ground Floor
Near Sukanti Devi Enclave, AIIMS Nagar Lane-1
Patrapada, Aaiginia, Bhubaneswar-751019, Odisha, India

1.3 Cautions For Use

- The product is for single use only.
- Users should consider the circumstances of use in deciding whether to remove any residual powder on gloves after donning. Powder can be removed by wiping gloves thoroughly with a sterile wet sponge, sterile wet towel or other effective method.
- Isolated cases of allergic reactions to latex product or powder have been reported. If you experience a reaction to this product, discontinue use immediately and seek medical help.

1.4 Storage Condition

- The atmosphere must always be conducive to the storage of the gloves.
- Gloves before and after opened for use shall be stored in dark, cool and dry place.
- Gloves need to avoid area with excessive heat. The storage areas need to be controlled below 40°C/104°F.
- Gloves need to be stored at the areas with good air ventilation.
- Gloves need to be shielded from direct sunlight and fluorescent lighting.
- Gloves need to be stored far away from any energy radiating and heat generating equipment.

1.5 Expiration Date

- The shelf life of the gloves is 3 years from the date of manufacture.